

# BC-T1<sup>+</sup>

3D RF & Cavitation with Vacuum System



**Operating Manual**  
© bio-curve corporation

✦ Please read through the operation manual before using the machine !

**MORE WE THINK, MORE WE DO !**



# CONTENTS

<b>1.APPLICATIONS</b> .....	<b>03</b>
<b>2.PANEL DESCRIPTION</b> .....	<b>04</b>
<b>3.HOW TO USE</b> .....	<b>05</b>
<b>4.OPERATION STEPS</b> .....	<b>06</b>
<b>5.WORK THEORY OF ULTRASONIC LIPOSUCTION EQUIPMENT</b> .....	<b>07</b>
<b>6.FUNCTION AND THERAPEUTIC THEORY</b> .....	<b>07</b>
<b>7. TECHNICAL ADVANTAGES</b> .....	<b>08</b>
<b>8.FEATURES</b> .....	<b>08</b>
<b>9.CAUTIONS</b> .....	<b>08</b>
<b>10.OPERATING IIIUSTRATION</b> .....	<b>09</b>
Massaging the body; .....	09
Massaging the back and arms; .....	09
Massing the abdomen; .....	10
Massaging Buttocks and legs .....	11
Massaging the back .....	11
<b>11.THE CLIENT AS FOLLOWING IS NOT SUITABLE FOR THE TREATMENT</b> .....	<b>12</b>
<b>12.ACCESSORIES</b> .....	<b>12</b>
<b>13.PRODUCT SERVICES AND MAINTENANCE</b> .....	<b>13</b>
<b>14.WARRANTY CARD</b> .....	<b>14</b>

## APPLICATIONS

---


Using RF wave heating deeply, When the skin deep collagen reaches 45–60°C, it will secrete new collagen, and rebuild skin soft support. At the same time, Provide oxygen and nutrients to skin. Strengthen cells, then firming skin, removing wrinkles, eliminating and softening cellular organization to achieve the goal of reducing weight.

1. Taking advantage of vacuum, adopting sucking, gliding, rhythm, lifting, make the muscle move. Can soften the fat tissue, eliminate the areolar tissue, Body shape, Lymph drainage, massage relaxing.

2. Using Red LED light can repair and improve the skin, strengthen the treatment effect, make skin much healthier.

40K cavitation using adopts the non-invasive shocking fat cavitation effect. Can crack the fat tissue and areolar tissue effectively, and the fat is metabolized and expelled from the urine. At the same time, it can also protect the neurohemal, make the best effect of weight loss.


## PANEL DESCRIPTION

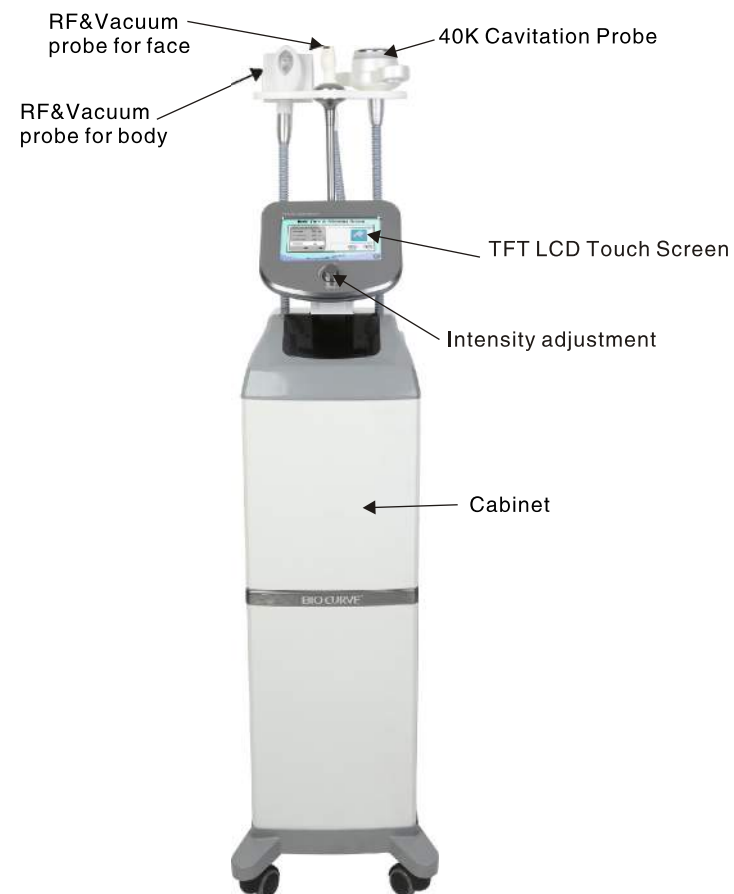

### Instrument feature

- ◆ Big touch screen, convenient operation
- ◆ Well appearance design
- ◆ Painless, safe ,and without anesthesia
- ◆ Non-invasive
- ◆ Easily realize gentle and graceful figure



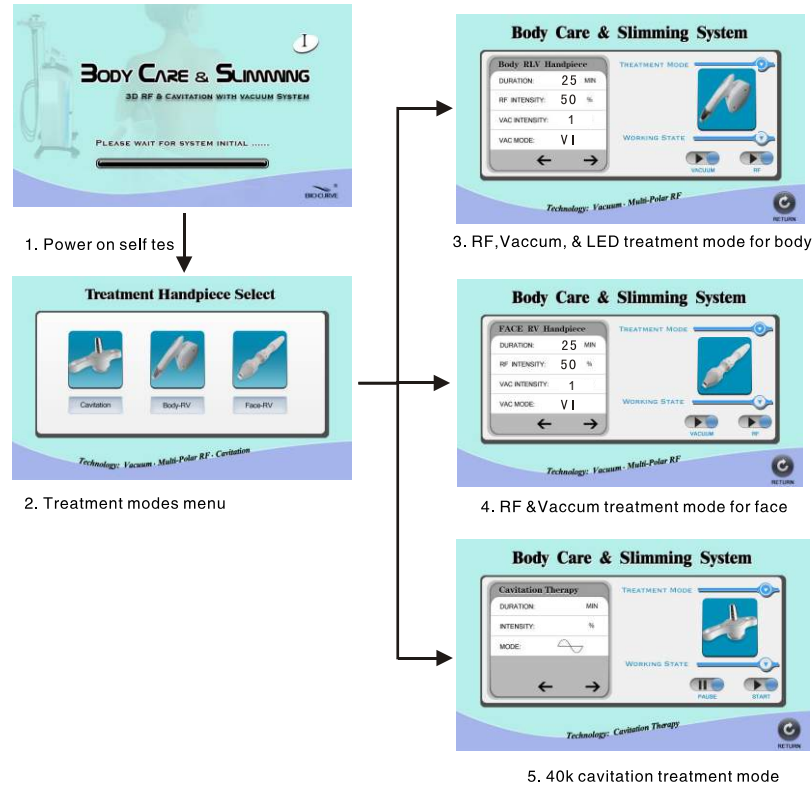

# HOW TO USE



## 1 Power on

Plug in, and turn on the power switch, the colored touch screen will display self test as picture 1. After finish, will enter the menu as picture 2. According to the treatment need of client, you can choose the treatment mode. Treatment modes include: 40K Cavitation, RF, Vacuum, & LED treatment for body, RF & Vacuum treatment mode for face. After choosing the treatment mode, will enter the accordingly working menu, you can start the treatment

## 2 Start/Pause

1. Enter into working menu. press "  ", it can work, display as "  " .
2. If you want to pause during the treatment ,press "  " again, it can stop. display as "  " .

# WARRANTY CARD

Maintaining sheet		
Product description :	Purchasing date:	
Outlet:	Add:	Tel.:
Client name:	Add:	Tel.:
Maintaining date	Fault case	Maintaining content

## PRODUCT SERVICES AND MAINTENANCE

1. The warranty date is one year from the date of purchasing. We are responsible for the repairing and the maintenance of the device. When the warranty date is expired, we will charge the service fee or the cost of the component. Please contact the distributor to purchase the extra accessory or the components.

2. We are not responsible for the free warranty as the situation as following:

- ◆ The failure caused by opening the machine without any authorization by the distributor or the manufacturer. So as modification of the product in private.
- ◆ The failure caused by dropping down the machine.
- ◆ The failure caused by unreasonable maintenance.
- ◆ The failure caused by incorrect operation, not following the direction of the instruction.

3. We keep the rights to modify the system without notice.

### ★ Statement

We are not responsible for any results caused by incorrect operation, please read though the instruction and then apply the operation.

### 3 TREATMENT PARAMETER MODIFY

**DURATION(TREATMENT TIME):**If you want to modify the treatment time, you should click the DURATION item first,and then press" → "to increase the treatment time," ← "to decrease the treatment time.Treatment time is 1–60mins.,STEPPING is 1 min

**RF INTENSITY:**If you want to modify the RF intensity,press the RF intensity item first,,then press" → " to increase the intensity, " ← "to decrease the intensity.Treatmnt intensity is from 10% to 100%.STEPPING is 10%

**VAC INTENSITY:**If you want to modify the Vacuum intensity,click the VAC INTENSITY item first.then press"→" to increase the intensity, " ← "to decrease the intensity.The scope of intiensity is 1 –5,STEPPING IS 1.

**VAC MODE :**If you want to modify the vacuum cycle,click the VAC MODE first,then press" → " to increase it, " ← "to decrease it.The scope of VAC MODE is "MODE-I、MODE-II、MODE-III、MODE-IV".

**INTERNSTIY of cavitation:**If you want to modify the intensity of caviation,click the intensity item first,then press" → " to increase the intensity ,and " ← "to decrease the intensity.The scope of intensity is 10%–100%.stepping is 10%.Cavitation intensity is for change the output cavitation intensity.

**Mode(cavitation mode):**If you want to change the cavitation mode,click the MODE item first.Then press " → " or " ← "to change the cavitation mode.including, continuous wave,Short pulsed wave.long pulsed wave.

## OPERATION STEPS

- 1、 Connect the power supply with the machine and the spare parts with the corresponding connector. Turn on the switch on the left side of machine body.
- 2、 Select the treatment mode.
- 3、 Press " ▶ " to start use.
- 4、 Press" ⏏ "to to stop use.

## THE CLIENT AS FOLLOWING IS NOT SUITABLE FOR THE TREATMENT.

- 1, People who have heart disease, cardiac rhythm device configuration, high blood pressure.
- 2, Patients who has acute inflammation, deep vein thrombosis, thyroid, asthma, cancer and other .
- 3, Vascular bleeding, and trauma, the wound bleeding people.
- 4, Symptoms of skin inflammation, edema of the human body.
- 5, During pregnancy or within 3 months postpartum, it is not recommended for treatment of this equipment.
- 6, The body treatment is required 1 hour after meals to avoid feeling discomfort.
- 7, Apparatus should be thoroughly cleaned after use, so that equipment will last long, It increases the life span of equipment.
- 8, Do not unauthorizedly demolish, repair, alter the equipment (This will lead the failure formation or the accident).
- 9, At the end of using equipment, be sure to turn off the power switch and unplug the power plug.
- 10, Pay attention to fire prevention, protection, moisture-proof, anti-collision of the product, accessories should be properly placed.

## ACCESSORIES

NAME	UNIT	QTY	PARTS NAME	UNIT	QTY
Main engine	PCS	1	Manual	PCS	1
Probe	PCS	3	Wire	PCS	1

## WORK THEORY OF CRYOLIPOSIS SUCTION EQUIPMENT

### 3D RF & Cavitation with vacuum slimming equipment

**RF:** use RF wave heating deeply, When the skin deep collagen reaches 45–60°C, it will secrete new collagen, and rebuild skin soft support. At the same time, provide oxygen and nutrients to skin. Strengthen cells, then firming skin, removing wrinkles, eliminating and softening cellular organization to achieve the goal of reducing weight.

**Vacuum** can treat promote siltation lymph, improve blood circulation, promote the absorption of exudate. Thus lead to the effect of lymph drainage and massaging

**LED:** LED can treat inflamed acne, Repair aging and sun burn skin, without any hurt and uncomfortable sense.

**CAVITATION:** cavitation transdermic's principle for treatment of adipose and cellulite. UltraLipo system combines the acoustic wave therapy, which achieves its almost incredible results by means of the effective impact of low-frequency, the pressure waves from AWT on localized fat tissues via acoustic cavitation and the subsequent reduction of the fat cells. The pressure also breaks up the collagen of the septae and releases to skin, allowing a smoother surface, and the fat is metabolized and expelled from the urine, from the urine.

## FUNCTION AND THERAPEUTIC THEORY

### 1. Mightily cracking the cellulite, remove the fatness.

It uses the strong ultrasonic head to shoot 40,000Hz ultrasonic, after entering the body, will make the fatty cells produce mighty cracking effect and fatty cells friction, effectively exhaust the heat energy, exhaust cells water, let fatty cells shrinking. Moreover, when the ultrasonic vibrates, there will bring mighty impaction among cells, fatty cells will be less, so get effect of fatness removal.

### 2. Dissolve the fatness, do lymph drainage, skin tightening, enhance skin elasticity.

It takes the most advanced RF technique and energy, directly penetrate into deep-seated cellulite. With targeted oriented RF output, it makes the fatty cells in quick active state, produces heat friction, higher the partial temperature. Through the sweat gland, liver sausage circulation and lymph, exhaust the redundant fatness and toxins out of the body, so get the effect of cellulite dissolving.

### 3. Improve the orange tissues, slim and sculpture the body contour.

Through the electrode energy of the equipment, it produces biology waves to stimulate the body relevant points. With different frequencies and pulses, under many physics electron reciprocity, stimulate the fatness, let the body exercise, so exhaust the heat energy and fatness again, get the effect of sculpturing the body.

## TECHNICAL ADVANTAGES

1. Adopt the top ultrasonic liposuction technique in the world.
2. Suitable for all kinds of skin.
3. Comfortable, painless, woundless during the treatment.
4. Easy operation, easy study.
5. Non- exhaustion, low cost investment and quick high returns.

## FEATURES

1. Free operation, Non-anaesthetic.
2. No ruggedness.
3. No bleeding, tumidness and bruise.
4. No side effects, good effects, no rebounding phenomenon.
5. Woundless, it will not influence the normal working and living.

## CAUTIONS

※ For reference, you can have your own operation method




Clean the lipid part of customer before the ultralipo treatment.

1、 Please coat the lipid part with professional cream before using the machine. The probe should never stay in the fat layer for long time and keep away from the prominent parts of the bones.

2、 Clean the skin with rinsing after treatment and ask the customer to lay down, resting for 5 minutes. We suggest the customer should drink 300 to 500CC warm water to promote the body metabolism, in order to facilitate the decomposition of waste toxins inside body

### ※ Massaging Buttocks and legs

1. back of thigh: Liquid drainage to the inguinal lymph nodes
2. calf: Liquid drainage to the inguinal lymph nodes;
3. joint: Joint liquid drainage to the lymph nodes;

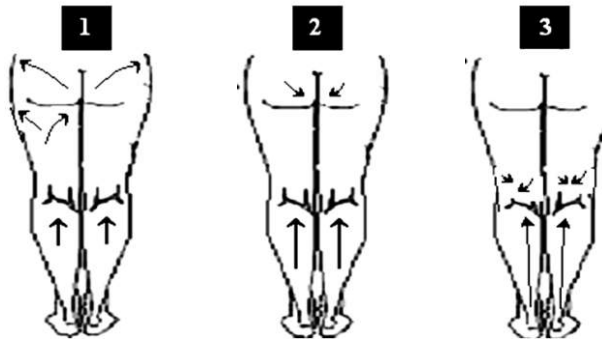

### ※ Massaging the back


1. Shoulder: The liquid drainage to the supraclavicular fossa lymph spaces
2. Hip: the liquid draining to the inguinal lymph nodes
3. Back: The liquid drainage to the axillary lymph nodes




## OPERATING ILLUSTRATION

### ※ Massaging the body;

- 1、Lifting with probe in orders : inner part of thigh → front part of thigh → outer thigh → groin ;
- 2、Operating the probe on outer thigh, pushing by “Z” or “8” –shaped;

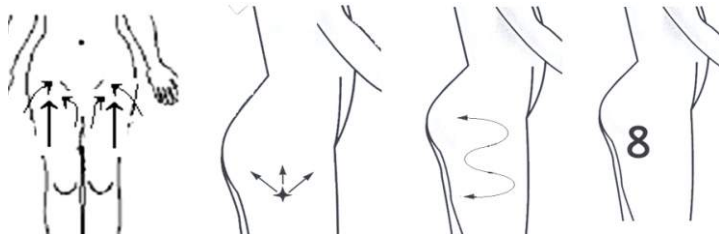

- 3、Lifting from the back of thigh to the buttocks upwards;
- 4、Lifting upwards or pushing by “Z” –shaped on the crus;

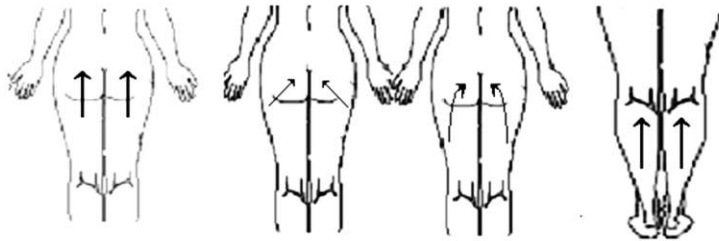


### ※ Massaging the back and arms;

- 1、Moving from the center of backbone to the lymph node in the armpit;
- 2、Moving from the back of arm to the lymph node in the armpit;
- 3、Moving from the front of arm to the lymph node in the armpit;



### ※ Massing the abdomen;

- 1、Moving from the center of backbone, through waist to the groin;
- 2、Moving in clockwise direction to the groin;
- 3、Moving downwards to the groin by “A” –shaped;

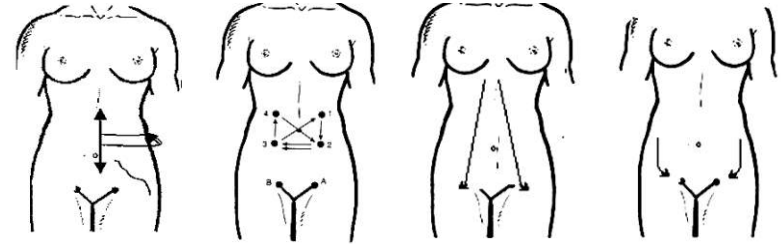

- 1、midsection: Liquid drainage to the axillary lymph nodes
- 2、bythus: Liquid drainage to the inguinal lymph nodes
- 3、waist: Liquid drainage to the inguinal lymph nodes

